CE

**EASYRED®**

EM185

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**CLOTH STRETCHERS  
INSTRUCTION MANUAL**

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I Class Medical Device  
according EEC directive 93/42

## Code EM185

Thank you for purchasing the **EASYRED** by Moretti cloth stretcher. This product is made of vinyl that can be cleaned and disinfected easily and it is used when the traditional rescue equipment can't be used; it is waterproof and washable.

Read this instruction manual before the use.

### Intended use

Device for rescue.

### Technical features

Length: 210x 75cm

Bag: 48 x 33cm

Max loading weight: 150 kg

### Instructions

1. Place the cloth stretcher in the floor.
2. Lay the patient on the stretcher.
3. Lift up the stretcher and transfer the patient in a proper place.

### Warning

1. Since the product is not rigid, don't use it for patient with fractures or spinal problems.
2. Six people can move the cloth stretcher.
3. Keep the product far from heat source.
4. Only qualified personnel can use this product and can assist the injured person; pay attention, above all, in case of spinal lesions.
5. Do not use this product for purposes not indicated in this manual.
6. Moretti SpA is not responsible for purposes not indicated in this manual.

### Maintenance and Cleaning

1. Use water and non aggressive disinfectant for the surface of the product; dry far from sun rays. Do not use clothes dryer or washing machine.

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## WARRANTY

This cloth stretchers has 2-YEAR warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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